

WHEELCHAIR SANITATION & DISINFECTANT INSTRUCTIONS

Safemark wheelchairs on your property should be regularly cleaned and maintained according to the instructions provided in the user manual. This process remains critical as items should always be thoroughly cleaned before being disinfected. Safemark specifies EcoLab Neutral Disinfectant Cleaner or Zep® Citrus Cleaner for everyday cleaning.



2019 NOVEL CORONAVIRUS SUPPLEMENT

Coronaviruses are a large family of common viruses. The 2019 Novel Coronavirus causes a disease that has been named "COVID-19". Information suggests that many illnesses associated with COVID-19 are mild, but severe illness can occur, especially among vulnerable populations such as the elderly and those with serious underlying health conditions. [The Centers for Disease Control have prepared a guide for keeping homes, schools and workplaces safe.](#)

Disinfecting using an EPA-registered product can be effective against the 2019 Novel Coronavirus; regular use offers a safer experience and reassures guests as our community deals with the impact of the current situation. Safemark is specifying the use of **Simple Green d Pro 5**, which has demonstrated effectiveness and can be used as a disinfectant, fungicide and virucide. This product has shown no detrimental effects to Safemark equipment in Safemark tests. [We have also identified a number of alternative disinfectant solutions if this product is unavailable.](#) **We recommend that wheelchairs be disinfected after each use.**



Simple Green d Pro 5
(EPA Registration Number
6836-140-56782)

PROCESS

Mix:
Simple Green d Pro 5 - 2 oz. (1/4 cup) : 1 gallon water

Spray & Wipe:
Use a coarse spray device held at least 6-8 inches from the surface to apply solution to the handles, armrests, frame and brake lever. Avoid breathing in the spray. Leave treated surfaces wet for 60 seconds. Wipe down with a clean cloth.

Spray & Let Dry:
Use a coarse spray device held at least 6-8 inches from the surface to apply solution to the wheelchair. Avoid breathing in the spray. Allowing the surface to air dry promotes maximum effectiveness.



ADDITIONAL BEST PRACTICES

Consider providing hand sanitizer stations and finger cots near high-touch areas, such as rental stations.

It is also safe and acceptable for guests to sanitize their wheelchairs using their own Lysol, Clorox or similar Disinfecting Wipes.



Please note: Rental Kiosks should be disinfected with an alcohol-based solution.

Please see unique instructions for Rental Kiosks.

CALL (TOLL-FREE)

(800) 255 8818

Select Option #2


EMAIL WeCare@safemark.com

VISIT safemark.com/cares

WHEELCHAIR SANITATION & DISINFECTANT INSTRUCTIONS

Be sure to clean wheelchairs thoroughly before beginning the disinfectant process.

Take special care when cleaning the high-touch areas highlighted in gold in the image below. Note that while the wheelchairs at your facility may differ in appearance from the model shown, the cleaning and disinfectant process remains the same.

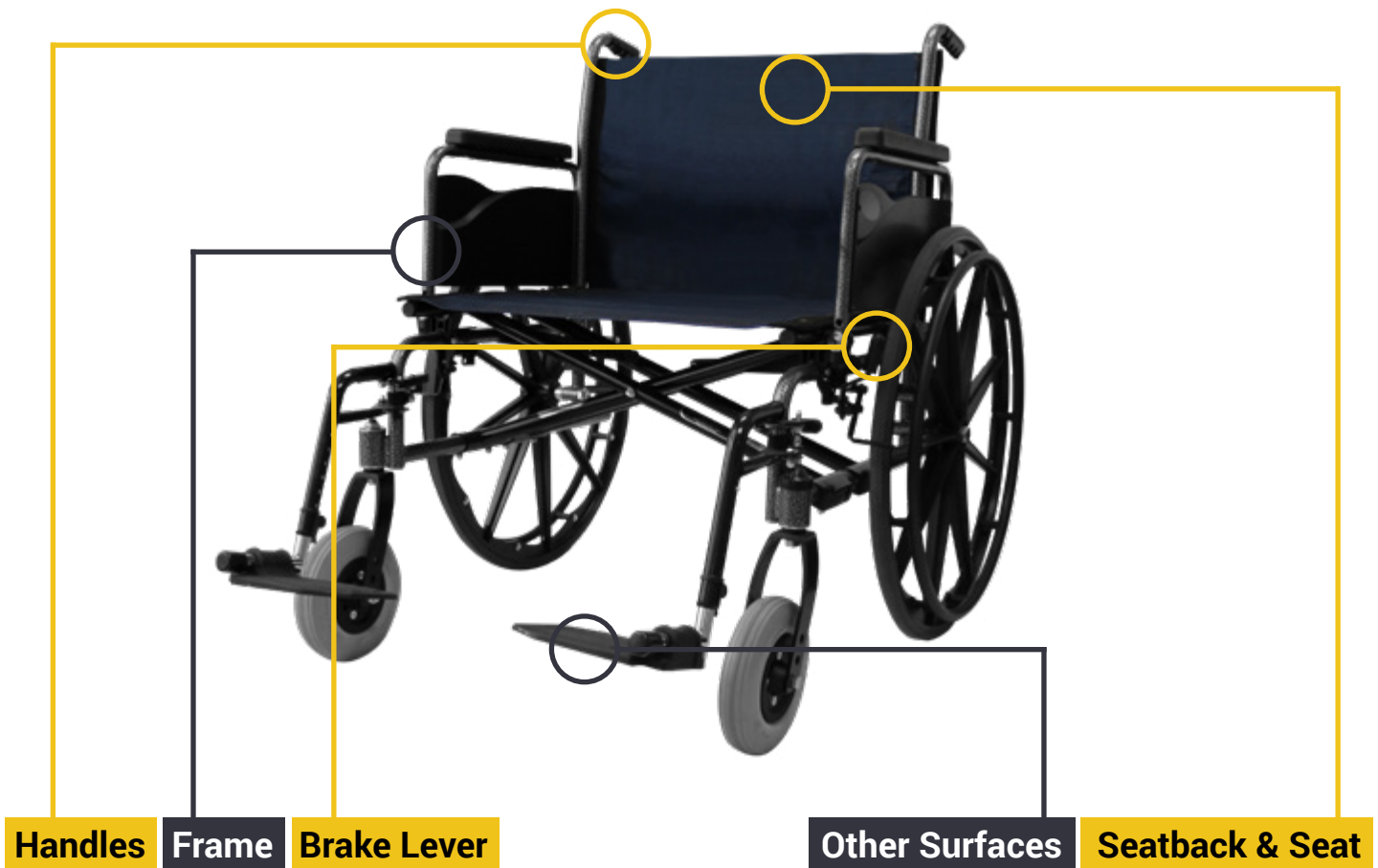

Spray & Wipe:

Mix 2 oz. (1/4 cup) Simple Green d Pro 5 cleaner with one gallon of water. Use a coarse spray device held at least 6-8 inches from the surface to apply solution to the wheelchair. Avoid breathing in the spray. Leave treated surfaces wet for 60 seconds. Wipe down with a clean cloth.

Spray & Let Dry: Apply cleaner using the same spray method as specified for the handles and armrests, frame and brake lever. Allow surface to dry.

QUESTIONS?

CALL (TOLL-FREE)

(800) 255 8818

Select Option #2

EMAIL

WeCare@safemark.com

VISIT

safemark.com/cares